Instructions for Use

TEMPLE TOUCH Temple & Underarm Thermometer PREMIUM

Digital Temple Thermometer


Thank you for purchasing the Temple Touch Premium Digital Temple thermometer.

This product enables accurate temperature measurements from the temple area or underarm in 6-8 seconds, using simple steps.

Table of contents:

	Safety warnings
Section 1:	Quick Start
Section 2:	measuring guidelines
Section 3:	Directions for use
Section 4:	Room Temperature Display
Section 5:	Using ODPT™ – Medicine Timer
Section 6:	Setting the operating scale °F or °C
Section 7:	Installing and replacing the batteries
Section 8:	About body temperature measurement
Section 9:	Troubleshooting and special indications
Section 10:	Care and cleaning
Section 11:	Technical Information
Section 12:	Warranty and Service Information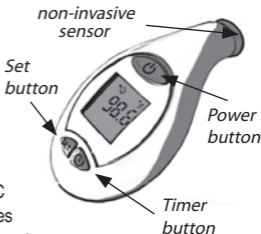

IMPORTANT! Read Instructions for Use before using the thermometer.

Safety warnings:

1. Keep the thermometer out of the reach of children.
2. Do not drop the thermometer or expose to heavy shock. The unit is not shock proof.
3. Do not immerse thermometer in water or any other liquid.
The unit is not water proof.
4. Do not store the unit in direct sunlight, at a high temperature, in high humidity or dust. Performance of the unit may be degraded.
5. Stop using the thermometer if it operates erratically or if display malfunctions.
6. Clean the thermometer probe before storing (see section 10).
7. Do not attempt to disassemble the unit except to replace the batteries.

1. QUICK START

1. **Press and release the POWER button.**
2. **Unit will beep twice when ready.**
3. **Verify temple or underarm mode: Push the SET button if you wish to change.**
4. **Hold firmly to temple or underarm, wait for long beep and read results.**

2. Measuring guidelines:

1. Measurement body location is the temple, in proximity to the temporal artery, between the end of the eyebrow and the hairline.
2. Do not place the thermometer on scarred tissue compromised by skin disorder, open

sores or abrasions and patients in trauma.

3. Temperature readings are affected by many factors including exercise and drinking hot or cold beverages prior to taking a reading.
4. Consult your physician when using drug therapies that may raise the local skin temperature in the forehead; this may lead to incorrect reading.
5. Do not place the thermometer on skin exposed to direct sunlight, fireplace heat, air conditioner flow and cold/warm compress therapies.
6. Wait at least two minute between consecutive measurements for the unit to readjust to room temperature.
7. If the thermometer has been stored in a cold temperature, allow the unit to warm naturally to room temperature before using.
8. Performance of the device may be degraded if operated or stored outside stated temperature and humidity ranges.


3. Directions for Use:

Use this thermometer on a clean and dry skin surface.

Wipe away any perspiration prior to use.

Taking Temperature:

1. Press and release the Power button, you will hear one short beep. All segments will be displayed (Figure 1) and the backlight will turn on automatically.
2. The previous result will be displayed for 2 seconds (Figure 2).
3. The room temperature and the hourglass will be displayed. (Figure 3).
4. Wait for the hourglass icon to disappear, then two beeps will be heard to indicate that the thermometer is now ready to measure.
Do not take a measurement before the two beeps are heard.
5. Verify that you are in the correct mode: temple or underarm (figures 4, 5), if you wish to change. Press and release the SET button.
6. **For forehead measurement:** Move hair aside from the temporal area and apply the probe to bare skin, in the center of the temple area, between the end of the eyebrow and the hairline (Figures 6a + 6b). Gently press and hold to ensure good contact with the skin is maintained.

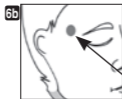Temple area

For underarm measurement:

- a. Place the thermometer probe in the center of the armpit as shown (figure 7).

Gently press the thermometer's probe in an upward motion.

- b. leave the patient's arm raised up until completed.

Correct placing and good contact with the skin ensure reliable readouts.

7. As soon as the thermometer is in place, the measurement will start automatically. A running circle will appear indicating that the measurement is progressing successfully (Figure 8).

Make sure that the thermometer and the patient do not move while measurement is in progress.

8. A temperature reading will be displayed within 6-8 seconds. One long beep will be heard to indicate that measurement is complete and temperature will be displayed for about 1 minute. The backlight will stay on until the device is shut down.
9. Press and release the Power button to turn the thermometer off, or it will shut off automatically within 1 minute.

If you need to take another reading – wait 2 minutes after turning the thermometer off and then begin again.

4. ROOM TEMPERATURE DISPLAY:

1. When the thermometer is OFF, continuously press the power button.
2. The last body temperature will be displayed, and then will switch to the room temperature. (see figure 9)

The room temperature will be displayed unless you press the power button again and switch off the thermometer.

5. Using ODPT™ – Over Dose Protection Timer:

The ODPT™ timer is designed to track the time elapsed between giving medicine dosages to patient and to act as an alert to parents as to when next medicine dosage should be given (based on time cycle set by parent).

How to Set & Start Timer

a. Setting- up the time cycle:

The ODPT™ timer is set for 6 hours by default.

To change the cycle (from 1-8 hours):

1. Switch OFF the thermometer.
2. Press and hold the SET button.
3. Continue pressing the SET button, then press the timer ⌚ button once and release both buttons.
4. The interval time value will blink on the display.
5. Press the "Timer" ⌚ button to scroll through time interval options. The display will show time values between 1-8 hours (1, 2, 3, 4, 5, 6, 7, 8) and then back to 1 hour (starting over again). (See figure 10).
6. Release the "Timer" ⌚ button when the desired value appears on the display. Then press the SET button in order to set the desired value.

b. Operating the timer:

1. To start the timer press and release the "Timer" button. The timer's display will show "0:00" (See figure 11) . Press again and the timer will start counting seconds until "0:01" minute and then minutes.
2. To reset the timer press and hold the "Timer" ⌚ button. (Press the "Timer" button until display changes to "0:00").
3. To start the timer again, press the "Timer" ⌚ button again. The count will start from "0:00".

IMPORTANT NOTES:

- The ODPT™ timer can be used while the thermometer is on or off.
- The ODPT™ timer continues to count the time elapsed even if the thermometer is turned off.
- It is not possible to turn on the timer when conducting temperature measurement. Temperatures can be taken while the timer is running.
- If the ODPT™ timer is stopped mid-cycle, cycle must be reset by pressing and holding the timer button for 3 seconds. To resume the timer press the timer ⌚ button again.
- When the time cycle has ended the thermometer will beep. The display will blink the pre-set time cycle for 60 seconds.
For example, if the timer was set to 6 hours, the image "6:00" will blink on the display.

6. Setting the operating scale °F or °C

The device setting is by default Fahrenheit (°F).

To change the scale to °C, or back to °F:

1. While OFF, press and hold the SET button.
2. Continue pressing SET, then press the POWER button.
3. Continue pressing both buttons until the scale changes and then release.
4. Wait for 2 beeps and measure temperature (see section 3)

7. Installing and replacing the Batteries:

The thermometer uses 2 standard AAA 1.5v batteries. To install/replace, first switch off the device:

1. Using a screwdriver, loosen the screw on the battery cover and lift it off the thermometer.
2. Remove the drained batteries and place new batteries in the battery compartment, matching polarity according to the diagram.
3. Secure the battery cover using the screw.


IMPORTANT: It is recommended to remove the batteries from the thermometer when it is not used for a long period.

WARNING: Do not dispose of batteries in fire, recharge, put in backwards, disassemble, mix with used or other battery types – they may explode or leak and cause injury. Waste batteries should be deposit in a batteries recycling trash only.

8. About body temperature measurement:

How does the thermometer work?

The accurate measurement is obtained by using a unique probe, which measures the heat flow from the blood vessels to the skin and converts it to body temperature.

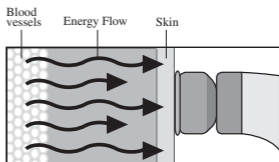Information about Body Temperature

What is “normal” body temperature?

“Normal” body temperature varies from person to person, changes according to time

of day and body location. Hot or cold baths, physical exercise, smoking and drinking, hot or cold drinks may change body temperature by one degree or more.

The following table shows the temperature ranges:

	What is fever?	Average normal temperature ranges
Forehead	Over 37.8°C/100°F	36.1°C–37.8°C / 97.0°F-100.0°F
Rectal*	Over 38°C/100.4°F	36.3°C–38°C / 97.3°F-100.4°F
Oral*	Over 37.8°C/100.0°F	36.1°C–37.8°C / 97.0°F-100.0°F
Axilla* (armpit)	Over 37.2°C/99.0°F	35.2°C–37.2°C / 95.4°F-99.0°F

*Fever definitions according to the American Medical Association

The thermometer is designed to measure the temperature in the forehead, in proximity to the temple area, between the eye socket and the hairline. It is important to know that the thermal characteristics of each measurement body location – temporal, axillar, oral or rectal - are unique. The variance in temperature measurement in different body locations is due to the human body and not the thermometer.

IMPORTANT: It is recommended to measure the body temperature when healthy to establish the specific “normal” temperature

9. Troubleshooting and special indications

The thermometer has been designed to display feedback-warnings to ensure proper usage and accurate temperature measurement. When an error appears, the device will "beep" 5 times, and then the thermometer will turn off automatically. In the event of a malfunction or incorrect temperature measurement, the following warnings will be displayed:

Indication:

The measured temperature is below 95°F (35°C) and the display presents: LO.

Instruction: 1. Turn the unit OFF and then ON again.

2. Retake temperature using proper placement technique (see section 3), while ensuring normal room temperature (73°F – 77°F / 23°C – 25°C).
3. If still reads LO, this indicates that the patient has a low body temperature.

Indication:

The measured temperature is above 107.6°F (42°C) and the display presents: HI

- Instruction:**
1. Turn the unit OFF and then ON again.
 2. Retake temperature using proper technique (see section 3).
 3. If still reads HI, this indicates that the patient has a high fever.

WARNING! A high fever or low body temperature requires seeking immediate medical attention**Indication 1:**

The thermometer is too cold or too hot to operate.

**Instructions:**

1. Verify room temperature within product range 60.8°F-104°F (16° C - 40° C)
2. Retake temperature using proper technique (See section 3).

Indication 2:

Temperature did not register because of poor contact, patient movement or poor placement.

**Instruction:**

Turn the unit OFF, wait 2 minutes, turn it ON again and retake temperature using proper technique.(See section 3).

Indication 3:

Unit malfunctioned.

**Instruction:**

Do not use the thermometer – contact service. (See section 9).

Indication 4:

Low battery - Battery icon is on continuously.

**Instruction:**

The battery has enough remaining life for approximately 10 more measurements

Indication 5:

Drained battery – only Battery icon is on when thermometer is turned on and the device turns itself OFF.

**Instruction:**

The battery is drained - replace battery.(See section 4)

10. Care and Cleaning

* Wipe the tip and probe with an alcohol swab or a cotton swab moistened with 70% isopropyl alcohol before & after each use.

After cleaning, wait at least 2 minutes before taking a measurement for the unit to readjust to room temperature.

* Use only a soft dry cloth to clean the body of the thermometer (if necessary). Never use an abrasive cleanser or submerge the thermometer in water or other liquids.

* Do not expose the thermometer to extreme temperatures, humidity, direct sunlight or shock. Store at room temperature.

11. Technical Specifications

Feature	Performance / Data
Measurement time	6-8 seconds
Measurement locations	Temple area of the forehead and underarm
Measurement range	95°F-107.6°F (35°C-42°C)
Operation environment	60.8°F-104°F (16°C-40°C) Humidity: 15-95% (non condensing)
Storage environment	-4°F-122°F (-20°C-50°C), Humidity: 15-95% (non condensing)
Measurement scales	Celsius or Fahrenheit
Accuracy	Complies with ASTM E 1112-00 standard requirements
Display	TN LCD 0.78" X 0.77" (20 mm X 19.6 mm)
Batteries	2 AAA 1.5volt batteries
Battery life	Minimum 1000 full duty cycles over 24 month period
Weight	2oz (56gr) including batteries
Dimensions	3.9" X 2.05" X 1.25" (10x5.2x3.2 cm)
Memory	Last reading display when turned on
Auto shut off	After 60 seconds
Case material	Food grade ABS plastic
Probe material	Food grade Stainless steel, silicone

12. Warranty and Service Information:

This thermometer has a one year warranty against manufacturers defect under normal use. Following the directions closely will ensure years of dependable operation. If the thermometer does not function properly, first check the batteries (see section 7). Replace if necessary.

If the thermometer does not function properly, please write to us at: customerservice@medisim-usa.com, or call us at 1-888-723-3659.

Our representatives will provide further instructions on how to correct the trouble or will ask you to return it for repair/replacement.

* Internally powered device

* Type - BF Applied part.

* Degree of protection against ingress of water: Ordinary

Distributed by: Medisim Ltd., G.G Communication Center, Neve ilan 90850 Israel.

Protected by US patents # 6,280,397 B1 and # 7,597,668 B2.

Made in China

Read the Instructions for Use

Applied part type BF equipment

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TEMPLE TOUCH Temple & Underarm Thermometer **PREMIUM**

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Distribuido por Medisim LTD., Centro de Comunicaciones GG, Neve Ilan 90850, Israel.