Instructions for Use

TEMPLE TOUCH


Temple Thermometer MINI

Thank you for purchasing the Temple Touch Thermometer.

Temple Touch enables accurate temperature measurements from the temple area in approximately 6-8 seconds, using simple steps.

Table of contents:

- Safety warnings**
- Section 1: Quick Start
 Section 2: measuring guidelines
 Section 3: Directions for use
 Section 4: Installing and replacing the batteries
 Section 5: About body temperature measurement
 Section 6: Troubleshooting and special indications
 Section 7: Care and cleaning
 Section 8: Technical Information
 Section 9: Warranty and Service Information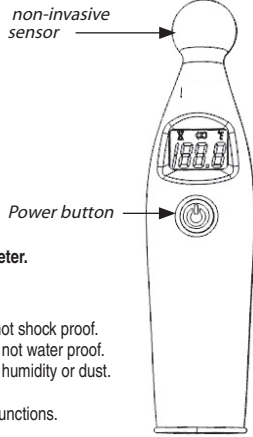

IMPORTANT! Read Instructions for Use before using the thermometer.

Safety warnings:

- Keep the thermometer out of the reach of children.
- Do not drop the thermometer or expose to heavy shock. The unit is not shock proof.
- Do not immerse thermometer in water or any other liquid. The unit is not water proof.
- Do not store the unit in direct sunlight, at a high temperature, in high humidity or dust. Performance of the unit may be degraded.
- Stop using the thermometer if it operates erratically or if display malfunctions.
- Clean the thermometer probe before storing (see section 7).
- Do not attempt to disassemble the unit except to replace the batteries.

1. QUICK START

- Press and release the **POWER** button.
- Unit will beep twice when ready.
- Hold firmly to temple, wait for next beep.
- Read results.

2. Measuring guidelines:

- Measurement body location is the temple, in proximity to the temporal artery, between the eye socket and the hairline.
- Do not place the thermometer on scarred tissue compromised by skin disorder, open sores or abrasions and patients in trauma.
- Temperature readings are affected by many factors including exercise and drinking hot or cold beverages prior to taking a reading.
- Consult your physician when using drug therapies that may raise the local skin temperature in the forehead; this may lead to incorrect reading.
- Do not place the thermometer on skin exposed to direct sunlight, fireplace heat, air conditioner flow and cold/warm compress therapies.
- Wait at least two minute between consecutive measurements for the unit to readjust to room temperature.
- If the thermometer has been stored at a cold temperature, allow the unit to warm naturally to room temperature before using.
- Performance of the device may be degraded if operated or stored outside stated temperature and humidity ranges.


3. Directions for Use:

Before measuring:

- Use this thermometer on a clean dry forehead. Wipe away any perspiration prior to use.
- Move hair aside from the temporal area and apply the probe to bare skin.

Taking Temperature:

- Press and release the Power button, you will hear one beep. All segments will be displayed (Figure 1)
- The previous result will be displayed for 2 seconds (Figure 2).
- Wait for the hourglass icon to disappear, then two beeps will be heard to indicate that the thermometer is now ready to measure (Figure 3).
Do not try to take a measurement before the two beeps are heard.
- Place the thermometer's probe in the center of the temple area between the end of the eyebrow and the hairline. Gently press and hold it against the temple to ensure good contact with the skin is maintained (Figures 4a + 4b).
Correct placing and good contact with the skin ensure reliable readouts.
- As soon as the thermometer is in place, the measurement will start automatically. A running circle will appear indicating that the measurement is progressing successfully (Figure 5). **Make sure that the thermometer and the patient do not move while measurement is in progress**
- A temperature reading will be displayed within 6-8 seconds. One long beep will be heard to indicate that measurement is complete and temperature will be displayed for about 1 minute
- Press and release the Power button to turn the thermometer off, or it will shut off automatically within 1 minute.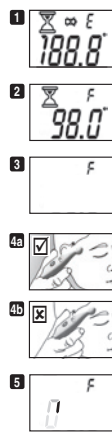


If you need to take another reading – wait 2 minutes after turning the thermometer off and then begin again.

Setting the operating scale °F or °C

The device setting is by default Fahrenheit (°F).

To change the scale to °C, or back to °F:

- Press and hold the Power button for 5 seconds – on the screen you will see the hourglass icon blinking and the last measured temperature (Figure 1).
- Once the last reading showed on the screen disappears, the scale icon (°F, °C) changes as well (Figure 2).
- Release the button, the hourglass icon will disappear in 5 seconds and 2 beep sounds are heard, indicating that the device is ready for use.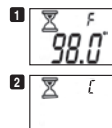

4. Installing and replacing the Batteries:

The thermometer uses 2 standard AAA 1.5v batteries. To install/replace, first switch off the device:

- Remove the battery cover by sliding it out of its housing.
- Firmly attach the batteries to their place, make sure polarity is correct. (see engraving on the bottom of battery compartment).
- Slide the battery cover back into place.


IMPORTANT: It is recommended to remove the batteries from the thermometer when it is not used for a long period.

WARNING: Do not dispose of batteries in fire, recharge, put in backwards, disassemble, mix with used or other battery types – they may explode or leak and cause injury. Waste batteries should be deposit in a batteries recycling trash only.

5. About body temperature measurement:

How does the thermometer work?

The accurate measurement is obtained by using a unique probe, which measures the heat flow from the blood vessels to the skin and converts it to body temperature.

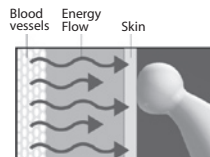Information about Body Temperature

What is "normal" body temperature?

"Normal" body temperature varies from person to person, changes according to time of day and body location. Hot or cold baths, physical exercise, smoking and drinking, hot or cold drinks may change body temperature by one degree or more.

The following table shows the temperature ranges:

	What is fever?	Average normal temperature ranges
Forehead	Over 37.8°C/100.0°F	36.1°C–37.8°C / 97.0°F-100.0°F
Rectal*	Over 38°C/100.4°F	36.3°C–38°C / 97.3°F-100.4°F
Oral (mouth)*	Over 37.8°C/100.0°F	36.1°C–37.8°C / 97.0°F-100.0°F
Axilla (armpit)*	Over 37.2°C/99.0°F	35.2°C–37.2°C / 95.4°F-99.0°F

* Fever definitions according to the American Medical Association

The thermometer is designed to measure the temperature in the forehead, in proximity to the temple area, between the eye socket and the hairline. It is important to know that the thermal characteristics of each measurement body location – temporal, axillar, oral or rectal - are unique. The variance in temperature measurement in different body locations is due to the human body and not the thermometer.

IMPORTANT: It is recommended to measure the body temperature when healthy to establish the specific "normal" temperature

6. Troubleshooting and special indications

The thermometer has been designed to display feedback-warnings to ensure proper usage and accurate temperature measurement. When an error appears, the device will "beep" 5 times, and then the thermometer will turn off automatically. In the event of a malfunction or incorrect temperature measurement, the following warnings will be displayed:

Indication: The measured temperature is above 107.6°F (42°C) and the display presents: HI

- Instruction:**
- Turn the unit OFF and then ON again.
 - Retake temperature using proper technique (see section 3).
 - If still reads HI, this indicates that the patient has a high fever.

WARNING! A high fever or low body temperature requires seeking immediate medical attention

Indication 1: The thermometer is too cold or too hot to operate.

- Instructions:**
- Verify room temperature within product range 60.8°F-104°F (16° C - 40° C)
 - Retake temperature using proper technique (See section 3).

Indication 2: Temperature did not register because of poor contact, patient movement or poor placement.

Instruction: Turn the unit OFF, wait 2 minutes, turn it ON again and retake temperature using proper technique. (See section 3).

Indication 3: Unit malfunctioned.

Instruction: Do not use the thermometer – contact service. (See section 9)

Indication: Low battery - Battery icon is on continuously.

Instruction: The battery has enough remaining life for approximately 10 more measurements

Indication: Drained battery – only Battery icon is on when thermometer is turned on and the device turns itself OFF.

Instruction: The battery is drained - replace battery.(See section 4)

7. Care and Cleaning

* Wipe the tip and probe with alcohol swab or a cotton swab moistened with 70% isopropyl alcohol before & after each use.

After cleaning, wait at least 2 minutes before taking a measurement for the unit to readjust to room temperature.

* Use only a soft dry cloth to clean the body of the thermometer (if necessary). Never use an abrasive cleanser or submerge the thermometer in water or other liquids.

* Do not expose the thermometer to extreme temperatures, humidity, direct sunlight or shock. Store at room temperature.

8. Technical Specifications

Feature	Performance / Data
Measurement time	6-8 seconds
Measurement locations	Temple area of the forehead
Measurement range	95°F to 107.6°F (35°C to 42°C)
Operation environment	60.8°F-104°F (16° C - 40° C) Humidity: 15-95% (non condensing)
Storage environment	-4°F to 122°F (-20° C - 50° C), Humidity: 15-95% (non condensing)
Measurement readout	Celsius or Fahrenheit
Accuracy	complies with ASTM E 1112-00 standard requirements
Display	TN LCD 0.63" X 0.44" (16 mm X 11 mm)
Battery	2 AAA 1.5volt batteries
Battery life	Minimum 1000 full duty cycles over 24 month period
Weight	1.3oz (37gr) including batteries
Dimensions	4.7" x 1.17" x 1" (12 cm x 3.0 cm x 2.4 cm)
Memory	Last reading display when turned on
Auto shut off	After 60 seconds
Case material	Medical grade ABS plastic
Probe material	Medical grade Stainless steel ,medical grade ABS plastic
Warranty	1 year

9. Warranty and Service Information:

This thermometer has a one year warranty against manufacturers defect under normal use. Following the directions closely will ensure years of dependable operation. If the thermometer does not function properly, first check the batteries (see section 4). Replace if necessary.

If the thermometer does not function properly, please call 1.888.723.3659 or visit us at www.medisim-usa.com.

* Internally powered device

* Type - BF Applied part.

* Degree of protection against ingress of water: Ordinary

Distributed by Wal-Mart Stores, Bentonville AR 72716.

Protected by US patent # 6,280,397 B1 and patent application # 11,444,710.

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Distribuido por Medisim LTD. G.G communication center